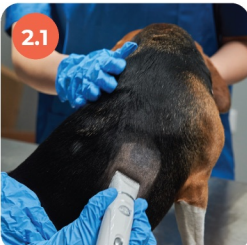
Quick Start Guide

1Download FurCarer APP

2.1

2.2

3.1

3.2

3.3

3.4

3.5

3.6

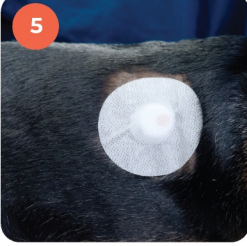
3.7

4

5

Start your Sensor

Instruction for Use

IEEnglish

1. Install the APP & Log In

Scan the QR code to download the FurCarer APP and register an account.

2. Prepare the Application Site

2.1 Shave the application site on your pet to remove fur. The cleaner the area, the better the sensor adhesion.
2.2 Clean the shaved skin using an alcohol pad to remove any dirt and oil. Allow the area to air dry completely.

We recommend placing the sensor on the upper back or side of the neck/shoulder blade area for dogs and cats, where it's harder for them to reach.

3. Apply the Sensor

3.1 Slide the orange button to the topmost position.
***Push upward until it can't go any further. You will see a silver seal pop out.**

3.2 Tear off the sealing ring and open the top cap to reveal the applicator button.

3.3 Firmly press the button until you hear a "click" sound.

3.4 Unscrew and remove the bottom cap.

***Do not press the applicator before placing it on your pet.**

3.5 Align the applicator vertically to the application site. Press the applicator firmly down **until you hear a "click."**

3.6 Gently lift the applicator straight off your pet's skin.
***Please do not discard the applicator now!**

3.7 Confirm that the sensor is attached and gently press around the adhesive area for about 10 seconds.

4. Pair the Sensor with the App

4.1 Use the APP to scan the QR code on the back of the applicator to connect the sensor.
4.2 If scanning fails, you may also enter the pairing code manually.

5. Apply the Overpatch

To ensure secure placement, apply the included overpatch on top of the sensor.
You'll find it in the envelope inside the product box.

We recommend extra protection for active pets — such as a snug-fitting pet shirt or a soft bandage wrap if needed.

Important:
Please review the complete instructions that come with your CGM device. These contain essential safety, usage, and care guidelines.
Visit [https://www.furcarer.com/downloads] or scan the QR code on the Quick Guide for the latest manuals.
You can also contact your local distributor for product support.

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FurCarer is a registered trademark of FurCarer Technology Co., Ltd.
All other product or company names that may be mentioned in this publication are tradenames, trademarks or registered trademarks of their respective owners.
HIPPO MEDICAL PTE. LTD.