FurCarer C0 Quick Start Guide

Start your Sensor

Instruction for Use IT English

- 1. Install the APP & Log In**
Scan the QR code to download the FurCarer APP and register an account.
- 2. Prepare the Application Site**
 - 2.1 Shave the application site on your pet to remove fur. The cleaner the area, the better the sensor adhesion.
 - 2.2 Clean the shaved skin using an alcohol pad to remove any dirt or oil. Allow the area to air dry completely.
- 3. Apply the Sensor**
 - 3.1 Slide the orange button to the topmost position. ***Push upward until it can't go any further. You will see a silver seal pop out.**
 - 3.2 Tear off the sealing ring and open the top cap to reveal the applicator button.
 - 3.3 **Firmly press the button until you hear a "click" sound.**
 - 3.4 Unscrew and remove the bottom cap. ***Do not press the applicator before placing it on your pet.**
 - 3.5 Align the applicator vertically to the application site. Press the applicator firmly down **until you hear a "click."**
 - 3.6 Gently lift the applicator straight off your pet's skin. ***Please do not discard the applicator now!**
 - 3.7 Confirm that the sensor is attached and gently press around the adhesive area for about 10 seconds.
- 4. Pair the Sensor with the App**
 - 4.1 Use the APP to scan the QR code on the back of the applicator to connect the sensor.
 - 4.2 If scanning fails, you may also enter the pairing code manually.
- 5. Apply the Overpatch**

To ensure secure placement, apply the included overpatch on top of the sensor.

You'll find it in the envelope inside the product box.

We recommend extra protection for active pets — such as a snug-fitting pet shirt or a soft bandage wrap if needed.

Important:
Please review the complete instructions that come with your CGM device. These contain essential safety, usage, and care guidelines.
Visit [https://www.furcarer.com/downloads] or scan the QR code on the Quick Guide for the latest manuals. You can also contact your local distributor for product support.

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